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REALMARE

Realmare spray

Seawater hypertonic solution for daily nasal and ear canal hygiene.

Realmare – Two products in one


Realmare is a hypertonic seawater solution that easily and effectively washes ears and nasal nostrils, thanks to its two specific sprayers. Both nose and ear are two anatomical “canals”, into which allergens, atmospheric dust and micro-pollutants can gather and cause problems.

Nasal hygiene:

Realmare spray is recommended in case of cold, sinusitis, and respiratory allergies of adults and children. Thanks to its hypertonic formula, rich in oligoelements, it is recommended for nasal congestion due to acute and chronic sinusitis, acute and chronic allergic rhinitis, vasomotor rhinitis, hypertrophic and infectious rhinitis. It removes excess mucus, clears nose from allergens, and favours proper breathing. It is also suggested for pregnant women.

Instructions for nasal hygiene:

Realmare spray can be used in any position with absolute safety.

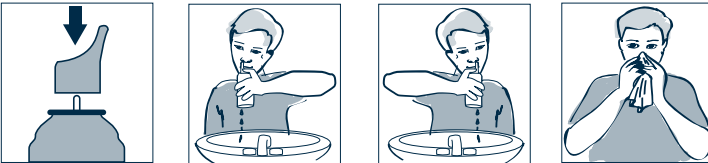- Apply the nasal nozzle on the bottle
- Insert the nozzle gently in the nostril.
- Press down firmly on the pump for a few seconds
- Blow your nose to eliminate mucous
- Repeat the process in the other nostril
- Wash thoroughly the nozzle with warm water after its usage

Posology for nasal hygiene:

Use Realmare spray whenever it is needed, with a frequency of 1 to 4 times a day for each nostril.

Ear hygiene:

Realmare has been studied to favour the removal of excess earwax, while helping prevent the formation of earwax plugs. Realmare is particularly suggested to people who play water sports, or wear hearing aids, and in case of excess ear wax.

Instructions for ear hygiene:

Realmare spray can be used in any position with absolute safety.

- Apply the auricular nozzle on the bottle.
- Tilt head to one side, insert the nozzle in the entrance of ear canal, and press down on the pump to obtain the desired quantity.
- Wait for a few seconds, to allow solution to exert its function, then return to vertical position and dry excess liquid.
- Repeat in the other ear canal.
- Wash thoroughly the nozzle with warm water after its usage

Posology for ear hygiene:

Use Realmare spray whenever it is needed, with a frequency of 1 to 4 times a day for each ear canal.

Ingredients: Maris Aqua, Aqua

Warnings: Do not use in children younger than 6 years. For nasal/auricular use only. Do not insert the nozzle deep into the ear canal. Avoid contact with eyes. In case of irritation, consult a doctor. Keep out of reach of children. Do not use after the expiry date printed on the bottom of the bottle. Do not use if the box or bottle are damaged. The product should never be used by more than one person. For correct use, follow carefully directions for use. Container under pressure: may explode if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Protect from sunrays. Do not expose to temperatures exceeding 50°C. Do not pierce or burn, even after use.

Revision 4 of January 2021

PNRM/PL/00111

3 270 REF PNRM001

: see under the bottle.

: see under the bottle.

EKUBERG PHARMA surl
Via Tito Schipa, 6
73020 Carpignano Salentino (LE) ITALY

CE 0546

Exclusive distributor for Greece:
LOGISCOOP PANSYFA AEE
DISTRIBUTION CENTRE FOR PHARMACEUTICAL
AND HEALTH CARE PRODUCTS
219 Gounari & 37 Tyrtaiou str., Glyfada
16674 Athens, Greece
www.logiscoop-pansyfa.gr

Legend of symbols:

Expiry date

Batch number

Manufacturer

See the package insert

Catalogue number